


*App features may vary

- **POWER ON TABLET:** Press power button behind tablet screen, to the right. Green light indicates it's powered on. No need to power off tablet
- **POWER ON SYSTEM:** Tap to wake App. Tap **Power icon** at App bottom right, and tap **Power On System**. System will launch into main "Levels" Portal
- **SELECT CAMERA VIEW:** Tap desired Camera view below **One-Touch Controls**. "Cameras" will show all camera views on Desk Display
- **ADJUST PAN + TILT:** Tap **Camera Joystick** at bottom left of App. Tap **name of camera** to adjust. Tap **arrows** or drag **center circle** to adjust
- **ADJUST ZOOM:** Use sliders to adjust zoom. Tap **Save icon** at right to save zoom as default. Tap **circular icon** to right of "Suite Levels" to return to default zoom
- **POWER DOWN SYSTEM:** Tap **Power icon** at bottom right, & tap **Power Down System**. Power down recommended nightly. No need to power off tablet

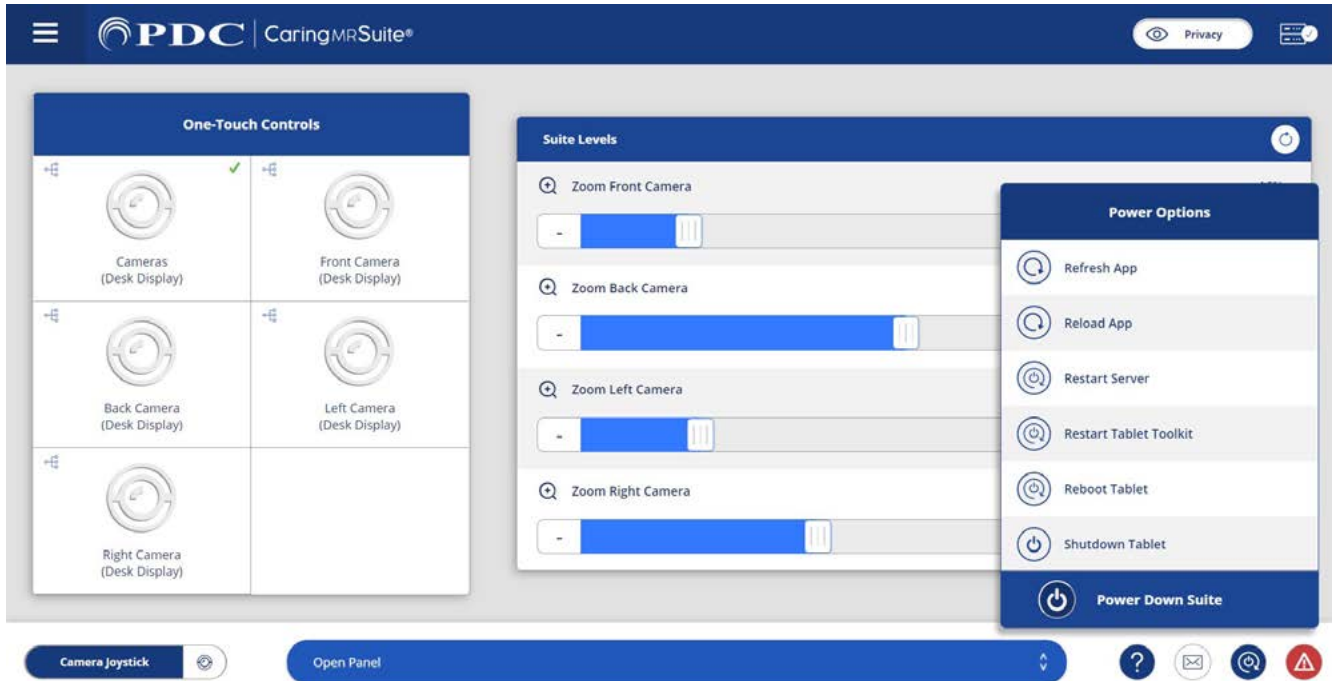


*App features may vary

TROUBLESHOOTING

- **APP ISSUES:** Tap **Power icon** at bottom right. First tap **Refresh App**. If needed, next tap **Reload App**
- **CAMERA/VIDEO ISSUES:** Tap **Power icon** at bottom right. Tap **Restart Server**
- **CONNECTION ISSUES:** Indicated by a red Rack icon in upper right. Tap **Power icon** at bottom right & tap **Restart Tablet Toolkit**. You can also tap **Reboot Tablet** or physically power off & on the tablet
- **NAVIGATING THE APP:** Tap 3-bar Menu icon in top left. "Levels Portal" contains camera controls. Admin Portal & Tablet Toolkit are for App setup. If within Tablet Toolkit, press **Caring Suite play button** to return to Levels Portal

Scan for Video Tutorials
& More Resources



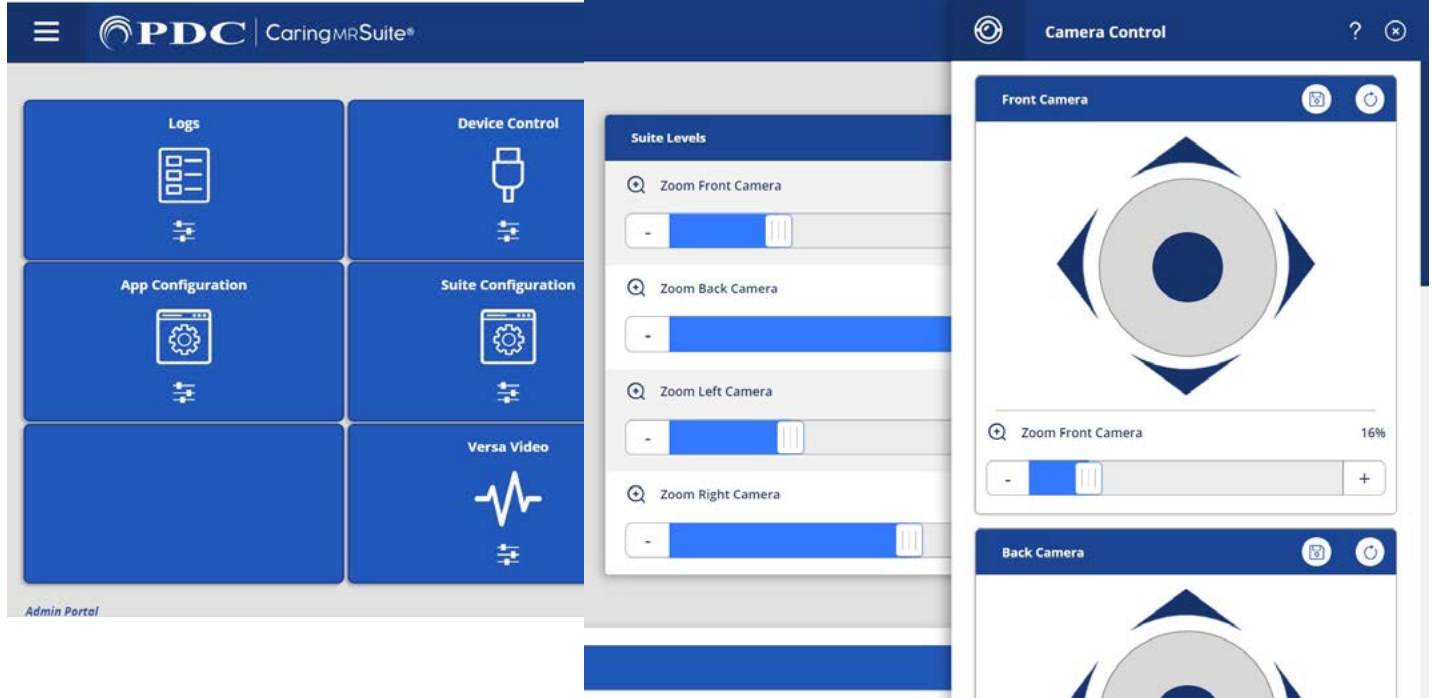

CUSTOMIZING APP

*App features may vary

- TO CHANGE CAMERA NAMES + SET DESK DISPLAY DEFAULT:**
 - Tap **3-bar Menu icon** in upper left, and tap **Admin Portal**. Enter: **8675309**
 - Tap **blue App Configuration** box
 - Below **Levels Portal** tap the **blue bar** beside **Main Action** & tap to select **Panel Links**
 - Tap **outside sidebar** to close
 - Tap **Menu icon** in upper left & tap to return to **Levels Portal**
 - Tap **Open Panel** at bottom center of App, & tap **Video Control**
 - Tap the **Gear icon** beside **Desk Display**
 - Locate **Camera options**, & tap **Pencil icon** to change camera name
 - Locate desired default option for Desk Display, i.e., **Cameras** to show all camera views
 - Tap the **Save icon** beside **Cameras** to save as default
- TO SET PAN-TILT-ZOOM DEFAULTS:**
 - Tap **Open Panel** at App bottom
 - Tap **Camera Control**. Adjust each camera's position and zoom
 - Tap **Save icon** to save as camera default
 - Tap **Circular icon** to return to default

Scan for Video Tutorials
& More Resources



Thank You!

We hope you love your new Pan-Tilt-Zoom Camera(s)!

For **Training** needs, please contact **Jessica** at jessica@pdcbiz.com