



Welcome To Your Suite!

Please select a role below.

PATIENT

TECHNOLOGIST

ADMINISTRATOR

App Version 20170926

Tap “Patient” for limited Patient options, or tap “Technologist” for full suite controls

“Patient” options shown below. Here, simply tap to select a Theme and Lighting color

SELECT YOUR THEME




Balloons



Big Fish



Children

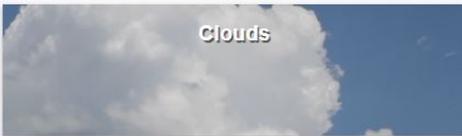

Clouds




Coastal Waters



Collage



Coral Aquarium

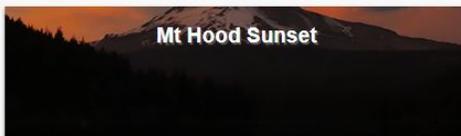

Mt Hood Sunset



Redwoods



Space




Tropics

Tap a Theme and Lighting Slider to choose your favorites!

SELECT YOUR COLOR



Color

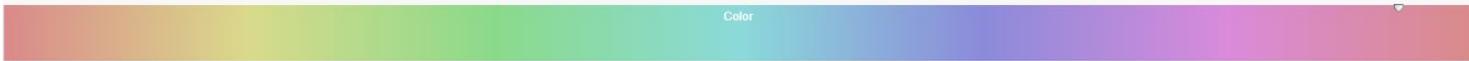

Find complete control of Suite video, music and lighting options here.

Tap “Select Theme” tab at top left to view Theme options. Tap to select Theme. Tap “Main Controls” at top left to return.

Technologist Controls:

System On/Off: Tap SYSTEM ON and SYSTEM OFF at top right

Emergency Mode: Tap to turn lights on bright and mute audio

Suite Settings: Options to customize suite and troubleshoot here

Lighting Color: Tap Color Wheel to choose a color. Use Color Square to modify color intensity

Video Source: Tap to select a Video Source. For DVD, next set-up DVD manually as usual, using remote to start video

Audio Source: Tap to select a source. For My Device, find audio jack, and setup music on device as usual. Ensure device volume is up, music playing

Display Rotation: Tap to change display orientation if needed. Often only “180” is needed. Tap “Normal” to return to feet-first orientation

Menu: Tap Menu icon at top left to return to Home and view more options.

Unsure of what is selected? Simply tap your desired option!

For Technical Support: 262-369-1929 techsupport@pdcbiz.com