ÔPDC CARING SUITE™ APP GUIDE

■ PDC CaringMRSuite [®]				App Configuration ? 💉
One-Touch Controls -4 -4 ViewRay M2 Standard (Portable Displays) ViewRay M2 Zoom (Portable Displays) -4 -4	Sound Control I〔」))) 幸	Video Control ▶ 추	Power Control	Tech Portal Is Default View: Yes Make Default One-Touch Configuration Configure Quick Select Configuration Configure
ViewRay M2 RealView (Portable Displays)	Display Control 다 호	Camera Contro ⓒ 후	テ Audio Channel Configure レ(しか) テ	Main Action Master Volume Slider Levels Portal Is Default View: No Make Default One-Touch Configuration Configure
- Themes				Quick Select Configuration Configure

EDITING OR ADDING A MAIN ACTION

- MAIN ACTION is an option that resides at App bottom center volume slider is shown above
- TO ADD OR EDIT A MAIN ACTION tap Menu icon at top left & tap "Admin Portal." Enter "8675309"
- TAP "APP CONFIGURATION" & find the Portal you wish to edit, i.e., "Tech Portal"
- TAP BLUE BAR BESIDE "MAIN ACTION" & tap to select your choice

