



RENAMING VIDEO DISPLAYS + SOURCES

- TO RENAME VIDEO DISPLAYS, i.e., change “Portable Display” to “Patient Display”:

 - Tap Menu icon in upper left, & tap “Admin Portal.” Enter passcode: “8675309”
 - Tap “Versa Video”
 - Tap “Display Groups” in left blue column
 - Tap Pencil icon beside Display name you wish to edit
 - Use keyboard to tap in new name

- TO RENAME VIDEO SOURCES, i.e., change “Portable Camera” to “Patient Camera”:

 - Tap Video Control Box
 - Tap Gear icon beside “Portable Display”
 - Tap Pencil icon beside the source you wish to edit