



PORTABLE VIDEO DISPLAY (PVD)

- **POSITIONING + MIRROR:** The PVD is most often positioned behind the bore & used with a Head Coil Mirror or other mirror solution, but can be brought to suite front
- **3 FEET:** The PVD is MRI-safe, but a 3-foot distance from magnet is advised
- **CABLE CAUTION:** PVD cable contains fiber optic cables; please ensure visitors don't step on or roll over cable, & store extra length in a figure 8 pattern, as above
- **ADJUSTING HEIGHT:** A small gray lever behind display offers height adjustment. Turn handle to unlock position & adjust to preferred height. Turn lever back to lock. A little muscle is all you need to adjust angle
- **180 ROTATION:** PVD can be rotated 180 degrees in either direction. Attempting to rotate 360 degrees may cause damage
- **PVD REMOVAL:** Please contact Tech Support 1st if you wish to remove PVD from suite. The System Control Rack's RED PDC drawer must be powered off 1st