Installation Guide

MRI In-Bore Viewing Mirror



Important Safety Instructions

When using electrical equipment, basic safety precautions should always be followed including the following:


Read And Follow All Safety Instructions

- 1. Warning: This luminaire must be installed in accordance with the NEC or your local electrical code. If you are not familiar with these codes and requirements, consult a qualified electrician. Attention: Ce luminaire doit être installé conformément au NEC ou à votre code électrique local. Si vous n'êtes pas familier avec ces codes et exigences, consultez un électricien qualifié.
- 2. **Warning:** This equipment is not for use in hazardous combustible atmospheres as defined by the National Electrical Code. *Avertissement: Cet équipement ne doit pas être utilisé dans des atmosphères combustibles dangereuses telles que définies par le National Electrical Code.*
- 3. **Danger**: Risk of shock Disconnect power before installation. *Danger: Risque de choc Couper l'alimentation avant l'installation.*


Save These Instructions For Future Reference

Installation:


Step 1: Place front foot into channel opening and slide the attachment into the rail of the table as shown in **Diagram 1**.

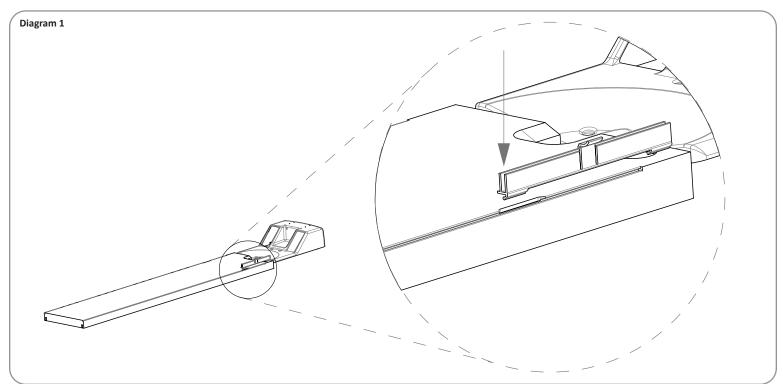

Step 2: Slide the attachment far enough into the table so the rear foot is not into the channel opening as shown in **Diagram 2**.

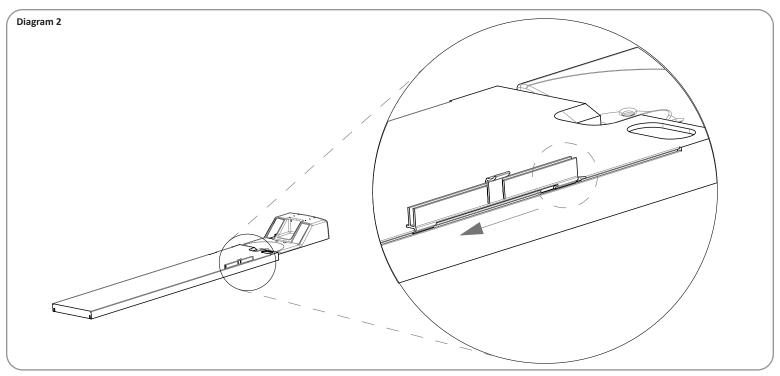

Step 3:
Slide the attachment towards the back of the table. It is now in the channel and can slide in the table for desired mirror position as shown in **Diagram 3**. Repeat for other side of table.

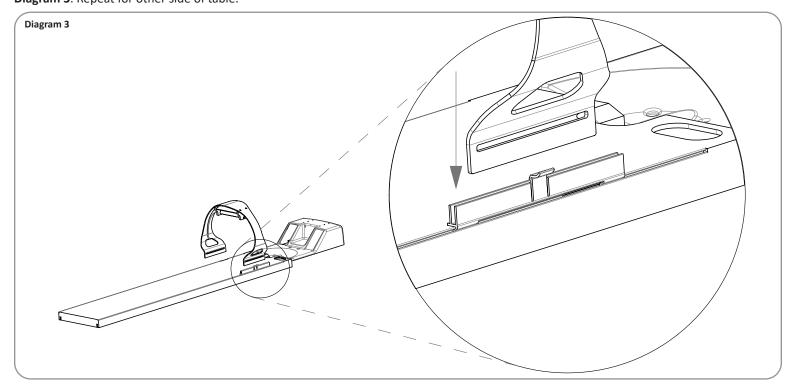



Directive 2012/19/EU (WEEE - Waste Electrical and Electronic Equipment): Information for Users

This product complies with EU Directive 2012/19/EU. The crossed out wheeled bin symbol on the article means that at the end of its useful lifespan, this product must not be disposed of with your other household or municipal waste. This product has required the extraction and use of natural resources for its production. It may contain hazardous substances that could impact health and the environment if not properly disposed. In order to avoid dissemination of those substances in our environment and to diminish the pressure on the natural resources, the user is responsible for delivering the appliance to an appropriate collection facility at the end of its useful lifespan. The separate collection and recycling of your waste equipment at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information on available collection facilities contact your local waste collection service or the place of purchase.



700 Walnut Ridge Drive Hartland, WI 53029 Phone: 262-367-7700 Web: www.pdcbiz.com Email: pdcbiz@pdcbiz.com