

PDC CARING SUITETM APP GUIDE








TABLET, AVB + MY DEVICE CABLE

- SUITE AVB (SHOWN ABOVE) BACK PORTS, from left to right:
 - Green: Not in use
 - Fiber Port: for Video communication between AVB & System Rack
 - Audio Input: For My Device
 - HDMI Input: For DVD player
 - Audio Output: For Patient Audio System, if applicable & integrated
 - HDMI Output: For any added displays
 - 19VDC: For Power cable
- AVB FRONT PORTS: Available for charging devices
- MY DEVICE CABLE:
 - Headphone jack with the white Apple adapter is My Device cable (See P7)
 - Shorter cables bundled with are for charging devices





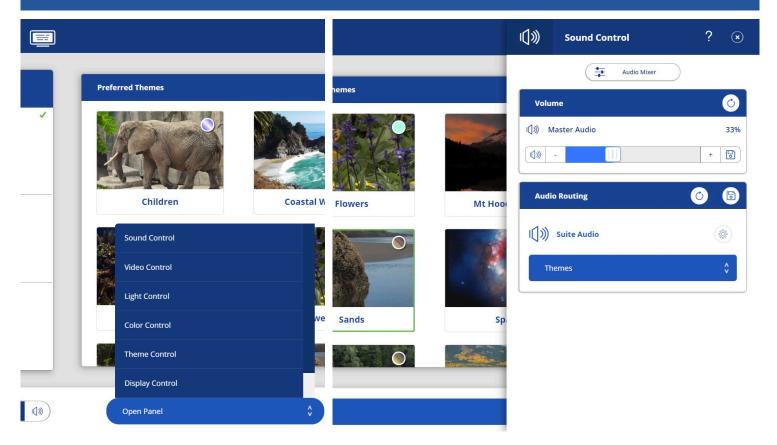

SOUND CONTROL

- TAP "OPEN PANEL" at bottom of App, & tap "Sound Control"
- CURRENT SOURCE: On blue bar below "Suite Audio" the current source is noted
- TO CHANGE SOURCE: Tap blue bar, & tap to choose new source. Standard sources include: Themes, DVD & My Device. THEMES features soothing music paired with Theme videos & MY DEVICE is a headphone-jack compatible music-playing device
- FOR MY DEVICE locate the system headphone jack, plug in music-playing device, ensure music is playing, volume is up & "My Device" is selected for Suite Audio
- TO QUICK-SELECT SOUND use One-Touch Control(s) (P5), Quick-Select Sound button at bottom left (P14) or "w/Sound" option in Video Control (P6), as applicable

