



TECHNOLOGIST PORTAL + DISPLAY CONTROL, P2

- **TO EDIT DEFAULT TIMERS & MESSAGES** tap “Expanded Display Controls” at top of Display Control
- **TO SAVE CUSTOM TIMERS & MESSAGES** tap the (+) icon on respective blue bars & enter desired Timers & Messages. For Messages, tap “Enter” to save
- **TO REMOVE TIMERS & MESSAGES** tap Pencil icon on the respective blue bar, & tap to remove
- **FOR TEMPORARY MESSAGES** tap “Load Custom Message.” These messages will not be saved. To remove, tap the Refresh icon or “Clear Patient Displays”
- **TO CHANGE DEFAULT DISPLAY ROTATION** tap “Set as Default” beside preferred Profile/orientation