

1




2



3




4

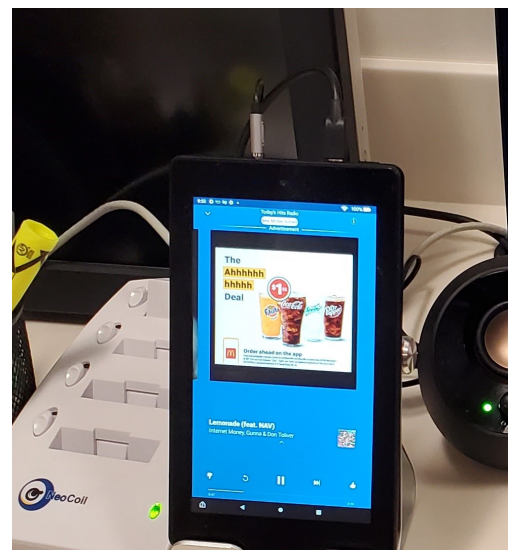

INTEGRATING WITH NEOCOIL PAS

TO INTEGRATE WITH NEOCOIL PATIENT AUDIO SYSTEM:

- Connect a 3.5mm to 3.5mm audio cable from the AVB "Audio Out" port (image 1) to the NeoCoil control box port below the music note (image 2)
- Connect the My Device headphone jack (image 3) to the NeoCoil-provided tablet (image 4) or other headphone-jack compatible music-playing device
- Unlink headphone volume from Master Volume Control, detailed on next page

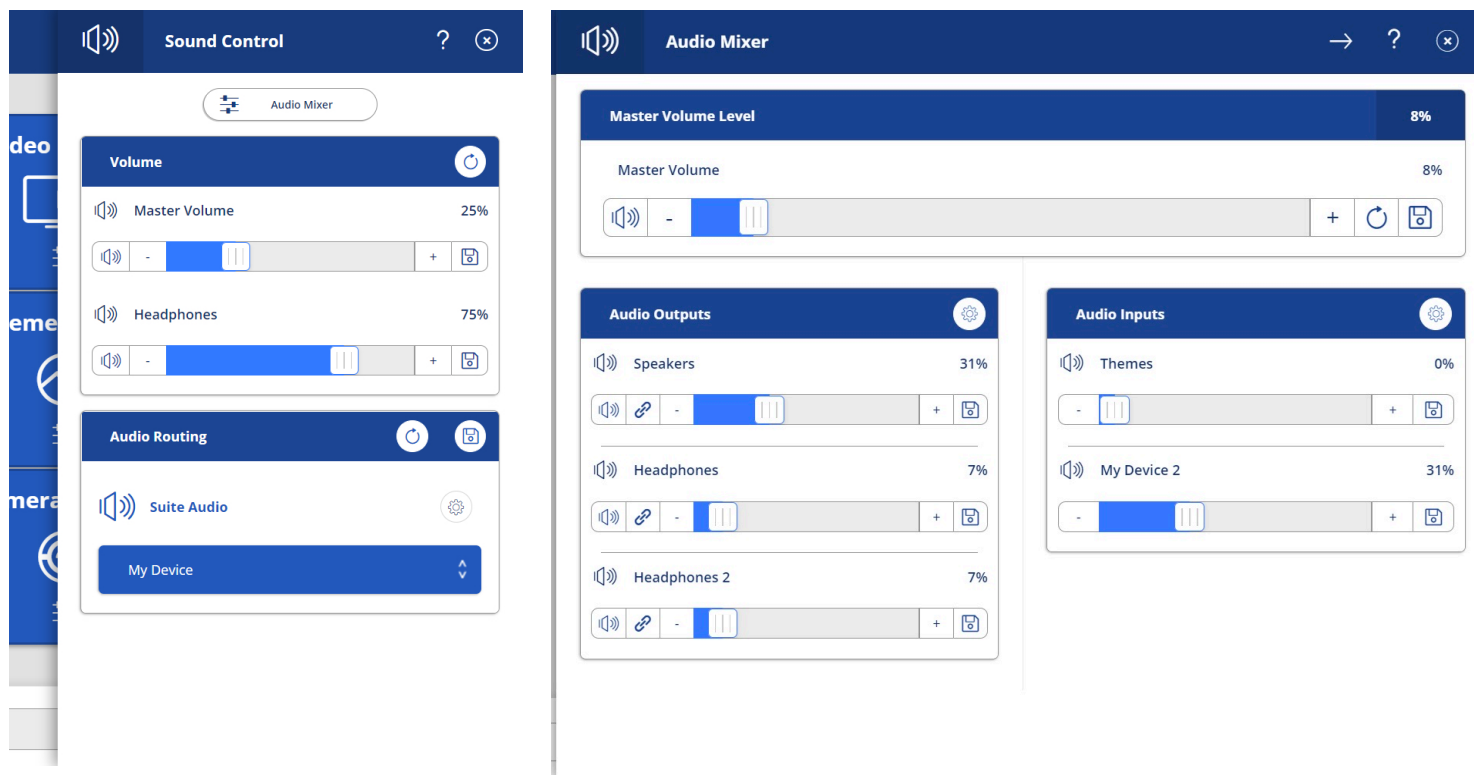

UNLINKING HEADPHONES FROM MASTER VOLUME

- **TAP BLUE SOUND CONTROL BOX** & tap “Audio Mixer” at top of side bar
- **TO UNLINK HEADPHONES FROM MASTER VOLUME:** Below “Headphones” in the “Audio Outputs” box, tap the blue link icon at left to unlink from the Master Volume. Link will change from blue solid to gray and broken. Unlinking from the Master Volume will allow you to control headphone volume independently of suite speaker volume, as shown in left image. Master Volume will now only control suite speaker volume.
- **SET HEADPHONE VOLUME** to a nice standard level. You can still fine-tune volume with your NeoCoil volume control